





How to install new 3Shape GenTek libraries

Step 1 delete all old GenTek libraries



- Open Dental System Control Panel
- Go to **Abutments** - Implant Systems
- Select old libraries
- delete select libraries
- Empty recycle bin
- Save changes

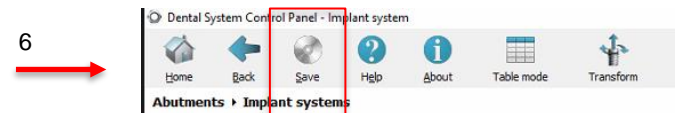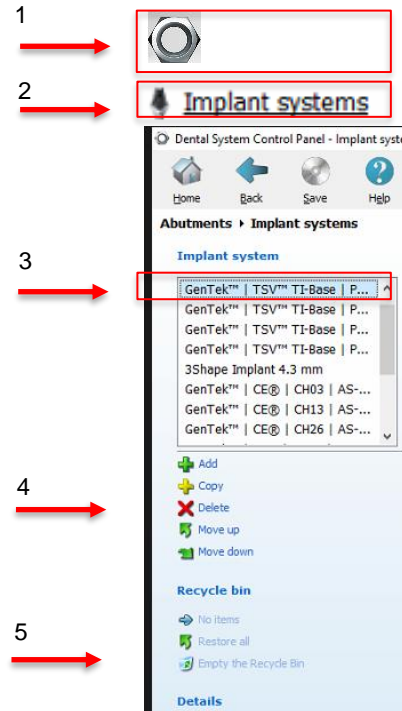

Step 2 import new GenTek libraries



- Turn back to Dental System Control Panel
- Go to **Tools** - Import/Export
- Select Import materials
- Import new library (dme)

1 →

2 →

3 →

4 →

5 →

6 →

7 →

- Save changes
- Close Control Panel and reopen Dental Manager, the libraries are now listed



Thank you

